

FORMOSTAR[®] ★

FORMOSTAR-Cardio

Stand 11.06.2019



TECHNICAL SPECIFICATIONS

Input voltage: 230/115 volts, 50/60 Hz.

Output voltage: 24 volts.

Power (standby): 11,8VA

Performance (1 bandage): 79.6VA

Performance (2 bandages): 158,8VA

Dimensions: W 50 cm x 50 cm (base plate), H 106 cm

FUNCTIONALITY

The device is placed next to or between two cardiometers. The customer operates the device itself as follows:

- **Remove the bandage from the device and change it (Velcro, self-explanatory)**
- **Press start button (The bandage heats up, 25-minute timer starts)**
- **Cardiom application is performed.**
- **After 25 minutes, the device switches off automatically - accompanied by an acoustic signal.**
- **Bandage is removed and hung over the holder on the device again.**

The device has two independent timers. Two applications can take place at the same time or with a time delay.